- EN SERVOX® Tracheal Compresses AL, standard, with punched hole, mini
- FR SERVOX® Compresses trachéales AL, standard, perforées, mini
- IT SERVOX® Compresse tracheale AL, standard, perforata, mini
- ES SERVOX® Compresas traqueales AL, estándar, perforadas, mini
- BG SERVOX® трахеален компрес AL, СТАНДАРТ, с отвор, mini
- HU SERVOX® standard légszó-sebpatna AL, lyukas, mini
- EL επίθεμα τραχειοστομίας SERVOX® AL, στάνταρ, με οπή, mini
- HR Trahealna kompresija SERVOX® AL, standardna, perforirana, mini
- NL SERVOX® tracheokompres standaard AL, geperforeerd, mini
- PT Compresse traqueal SERVOX® AL standard, perfurada, mini
- PL Kompres tracheostomijny SERVOX® AL standard, z otworami, mini
- SL Standardna trahealna kompresija z luknjico SERVOX® AL, mini

**DE** Nicht wiederverwenden – Infektionsgefahr  
**EN** Do not reuse – risk of infection  
**FR** Ne pas réutiliser – risque d'infection  
**IT** Non riutilizzare – rischio di infezione  
**ES** No reutilizar – riesgo de infección  
**BG** Да не се използваш повторно – опасност от инфекция  
**HU** Ne használja újra – Fertőzésveszély  
**EL** Μην επαναχρησιμοποιείτε – Υπάρχει κίνδυνος μόλυνσης  
**HR** Nemojte ponovno upotrebljavati – opasnost od infekcije  
**NL** Niet hergebruiken – risico op infecties  
**PT** Não reutilizar – Perigo de infeção  
**PL** Nie używać powtórnie – ryzyko infekcji  
**SL** Izdelek ni primeren za ponovno uporabo – obstaja nevarnost okužbe

**Inhalt / Contents / Contenu**  
 Contenu / Contenido  
 съдържание / tartalmat  
 περιεχόμενα / sadržaj / inhoud  
 conteúdo / zawartość / vsebine

**10 Stück**  
 pcs. | pcs. | pz. | pza. | броя  
 darab | τεμάχιο | kom. | stuks  
 unidades | szt. | kosov

**PZN 03802734**

**REF**  
**18008**

**Servona GmbH**  
 Biberweg 24-26  
 53842 Troisdorf | Germany

Tel. +49 (0)2241 9322-0  
 Fax +49 (0)2241 9322-277  
 info@servona.de | www.servona.de

18015/Version 2022-01-20

- DE **Größe**
- EN Size
- FR Taille
- IT Dimensioni
- ES Tamaño
- BG размер
- HU méret
- EL Μέγεθος
- HR veličina
- NL maat
- PT Tamanho
- PL rozmiar
- SL velikost

**65 x 70 mm**

- DE **Farbe**
  - EN Colour
  - FR Couleur
  - IT Colore
  - ES Color
  - BG ЦВЯТ
  - HU szín
  - EL χρώμα
  - HR boja
  - NL kleur
  - PT cor
  - PL kolor
  - SL barva
- DE **weiß**
  - EN white
  - FR blanc
  - IT bianco
  - ES blanco
  - BG бяла
  - HU fehér
  - EL λευκή
  - HR bijela
  - NL wit
  - PT branco
  - PL kolor biały
  - SL bele barve

- Material **DE Saugfähige, atmungsaktive Viskose, einseitig aluminiumbedampft**
- Material **EN Absorbent, breathable viscose fibre, aluminium coating on one side**
- Matériel **FR Fibre de viscose absorbante et respirante, métallisée aluminium d'un côté**
- Materiale **IT Fibra di viscosa assorbente, traspirante, alluminata su un lato**
- Material **ES Fibras de viscosa transpirables y absorbentes con una cara metalizada de aluminio**
- Материал **BG Хигроскопични, активно дишащи двуслойни вискозни влакна, алуминизирани от едната страна**
- Anyag **HU Nedvszívó, lélegző, kétrétegű viszkóz szál, egyik oldalán alumínium-réteggel**

- Υλικό **EL Απορροφητικές, αναπνεύσιμες ίνες βισκόζης διπλής στρώσης, με αναπόθεση ατμών αργιλίου στη μια πλευρά**
- Materijal **HR Dva sloja upijajućih, prozračnih viskoznih vlakana, aluminizirana s jedne strane**
- Materiaal **NL Absorberende, ademende, dubbellaagse viscose, aan één kant gealuminiseerd**
- Material **PT Fibras de viscose absorventes, que promovem a respiração, de camada dupla, revestidas a alumínio de um lado**
- Material **PL Chłonne, oddychające, dwuwarstwowe włókna viskozowe, z jednostronną cienką powłoką aluminiową**
- Material **SL Vpojna, zračna, dvoslojna vlakna iz viskoze, na eni strani aluminizirana**

**DE**

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**EN**

**Intended purpose** SERVOX® Tracheal Compresses AL standard, with punched hole, mini provide cushioning for the neck flange and absorb moisture that secretes from the tracheostoma to prevent skin irritation and infections.

**Indication** Tracheostomised or laryngectomised tracheal cannula users with normal secretions.

**Description** The aluminium coating helps prevent the compressa from sticking to the skin. The punched hole ensures a more secure fit and prevents slipping.

**Application** Slide the tracheal cannula with the aluminium coating to the tracheostoma through the punched hole of the compress until the neck flange is well cushioned. Then insert the tracheal cannula as usual.

To remove, pull out the tracheal cannula as usual, pull the compress off the cannula tube and dispose of it. – **Changing frequency:** with every cannula more frequently as needed, at least every 24 hours. – If the skin is irritated, use a compress with no aluminium coating and consult a doctor.

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