## Tracheo Tape soft plus

<b>DE</b>	<b>Gepolstertes Kanülentrageband mit Haken (weiß)</b>
<b>EN</b>	<b>Padded tube holder with hooks (white)</b>
<b>FR</b>	<b>Bande rembourrée porte-cannule avec crochet (blanche)</b>
<b>IT</b>	<b>Nastro portacannula imbottito con gancio (bianco)</b>
<b>ES</b>	<b>Cinta de fijación para cánula, acolchada, con gancho (blanca)</b>
<b>NL</b>	<b>Ge vulde canuledraagband met haakjes (wit)</b>
<b>EL</b>	<b>Φέρουσα κορδέλα σωλήνα με μαλακή επένδυση και γάντζο (λευκή)</b>
<b>BG</b>	<b>Поддържаща лента за канюли, с кукички (бяла)</b>
<b>HR</b>	<b>Pojastučena fiksacijska traka za kanile s kukicama (bijela)</b>
<b>PL</b>	<b>Opaska do kanili tracheostomijnych z miękkim poszyciem i z haczykiem (kolor biały)</b>
<b>PT</b>	<b>Fita de suporte de cânula almofadada com gancho (branco)</b>
<b>SL</b>	<b>Oblazinjeni nosilni trak za kanilo s kavlji (bele barve)</b>
<b>HU</b>	<b>Párnázott kanültartó pánt kapocsal (fehér)</b>

**REF** XXXXX

**LOT** XXXXXX

**PZN** xxxxxxxx

yyyy-mm

**UDI** (xx)xxxxxxxxxxxxxx

(xx)xxxxxxxx

**Orbisana Healthcare GmbH**  
Biberweg 24-26 | 53842 Troisdorf | Germany  
Tel. +49 (0)2241 9322-0  
info@orbisana.de | www.orbisana.de

41252/Version 2023-11-30

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**EN Intended use | indication | description**  
The SERVUX® Tracheo Tape soft plus is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open.  
The SERVUX® Tracheo Tape soft plus secures the aid in the tracheostoma and prevents it from accidentally slipping out.  
• Reverse side of tube holder with cotton coating  
• Suitable for all age groups depending on the neck circumference  
• single-use, do not reuse (risk of infection)  
• The maximum duration of use is 48 hours, then dispose of in household waste  
• Can be seamlessly adjusted to the circumference of the neck  
• Soft padding to prevent decubitus

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## EN Application

- 1 Place the band around the back of the neck
- 2 Insert both hooks into the eyelets on the neck flange from below. The back of each hook is facing the throat.
- 3 Use the slider to adjust the length of the band.
- 4 Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- 5 To remove, unhook one side of the band from the neck flange.

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## EN Please note

- Regularly check the fit and adjust where necessary.
- Remeasure the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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