SERVOX®

Tracheo Tape M soft plus

DE	Gepolstertes Kanülentrageband mit Klettverschluss (weiß)
EN	Padded tube holder with Velcro (white)
FR	Bande rembourrée porte-canule à fermeture auto-agrippante (blanche)
IT	Nastro portacannula imbottito con velcro (bianco)
ES	Cinta de fijación para cánula, acolchada, con cierre de velcro (blanca)
NL	Gevulde canuldraagband met klittenbandsluiting (wit)
EL	Φέρουσα κορδέλα σωλήνα με μαλακή επένδυση και Velcro (λευκή)
BG	Поддържаща лента за канюли, с подплата, с велкро закопчалка (бела)
HR	Pojastučena fiksacijska traka za kanile s čičkom (bijela)
PL	Opaska do kaniuli tracheostomijnych z miękkim poszyciem, zapinana na rzepy (kolor biały)
PT	Fita de suporte de cânula almofadada com fecho de velcro (branco)
SL	Oblazinjeni nosilni trak za kanilo s sprijemalnim zapiralom (bele barve)
HU	Párnázott kanültartó pánt tépőzárral (fehér)

REF XXXXX **LOT XXXXXX**

PZN xxxxxxxx **yyyy-mm**

UDI

(xx)xxxxxxxxxxxxxx
(xx)xxxxxx
(xx)xxxxxx

Orbisana Healthcare GmbH
Biberweg 24-26 | 53842 Troisdorf | Germany
Tel. +49 (0)2241 9322-0
info@orbisana.de | www.orbisana.de

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EN Intended use | indication | description

The SERVOX® Tracheo Tape M soft plus is used in laryngectomized and tracheotomized patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open. The SERVOX® Tracheo Tape M soft plus secures the aid in the tracheostoma and prevents it from accidentally slipping out.

- Reverse side of tube holder with cotton coating
- Suitable for all age groups depending on the neck circumference
- Single use, do not reuse
- The maximum duration of use is 48 hours, then dispose of in household waste
- Can be seamlessly adjusted to the circumference of the neck
- Soft padding to prevent decubitus

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EN Application

- 1 Open all Velcro fastenings.
- 2 Place the band around the back of the neck.
- 3 Slide both ends of the band through the eyelets on the neck flange from below and close the Velcro. The Velcro must be on the outside.
- 4 Use the Velcro fasteners to adjust the length of the band.
- 5 Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- 6 To remove, release the Velcro on one side and pull the band out of the eyelet.

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