SERVOCURA®

2-Wege Ballonkatheter

Silikon | TIEMANN SPITZE | NELATON SPITZE

2-way Foley Catheter

Silicone | TIEMANN TIP | NELATON TIP

DE

EN

Orbisana Healthcare GmbH

Biberweg 24-26
53842 Troisdorf | Germany
Tel. +49 (0)2241 9322-0
Fax +49 (0)2241 9322-277
info@orbisana.de
www.orbisana.de

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Symbols used

8

This Instructions for Use is intended for patients, relatives and healthcare professionals. SERVOCURA® Foley Catheters are indicated for use at home and in clinical settings for all age groups. They are designated for use by physicians or healthcare professionals trained in the use of bladder catheters and must be applied in accordance with the descriptions in this Instructions for Use and with recognized rules of medicine. Keep this Instructions for Use in a safe place for future reference. The sale of this product is restricted by or on the order of a physician.

SERVOCURA® Foley Catheters are assigned a lot number for unique traceability. The lot number is marked on the packaging following the „LOT“ label. In case of questions or problems with this product, please contact your sales representative or Orbisana Healthcare GmbH by stating the lot number.

EN

1. Intended Use and Indication

By inserting the catheter through the urethra into the bladder, the SERVOCURA® Foley Catheter helps patients suffering from dysuria to empty the bladder smoothly. Furthermore, it also helps incontinent patients to control the emptying of the bladder. Moreover, the product can also be used for urine samples and to collect blood clots or sediment from the bladder after surgery.

2. Contraindication

- Acute urethritis
- Acute prostatitis
- Acute epididymitis
- Patients with silicone allergy
- Patients with urinary stones
- Bleeding in the urinary tract, injury to the urethra

3. Restrictions

Do not use during an MRI (Magnetic Resonance Imaging) scan. Patients during the menstrual period are at increased risk of infection. People with impaired consciousness should be carefully monitored, unintentional pulling of the catheter may cause bladder and urethral injury.

Note

SERVOCURA® Foley Catheters are intended for single use only and may not be reused. There is a risk of infection.

4 Side Effects and Complications

- Nosocomial urinary tract infection
- Mucosal injury or bleeding of the urethra
- Shock to the patient
- Temporary sexual dysfunction
- Poor urinary drainage
- Urine leakage
- Bladder spasm or atrophy
- Urinary retention
- Formation of a shrinking bladder
- Difficult catheter removal

42921/Version 2024-03-13

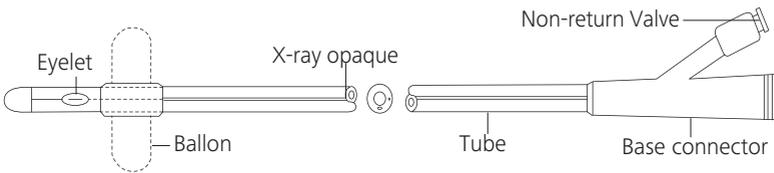
5 Safety Instructions

- Do not use for more than maximum 29 days.
- For long-term use, clean the urethral opening daily, replace the urine bag and possibly flush the bladder to prevent retrograde urinary tract infection.
- Do not use if the package or catheter are damaged or contaminated.
- Do not use if the balloon is difficult to be blocked.
- Do not continue to insert the catheter if it cannot be easily pushed through the urethra.
- Do not fill the balloon beyond the specified volume (10 ml).
- Never fill the balloon with any saline, air or contrast medium.
- When using a guide wire, ensure that it is attached to the front of the catheter. Avoid pulling and pushing movements when the guide wire is not in the eyelet to avoid injury to the urethra. Never move the guide wire intentionally.
- Never remove the colored sleeve of the check valve.
- Unintentional pulling of the catheter can cause bladder and urethra injury.
- Do not use animal or vegetable fats or oils (e.g. olive oil, white Vaseline, mineral oil) as lubricants.
- Never insert the catheter too quickly when the bladder is full. This may cause bleeding or shock to the patient.

Note

Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if in-patient treatment is required or must be prolonged because of its use.

6 Product Description

The SERVOCURA® Foley Catheter is a 2-way catheter made of silicone, with a blocking balloon behind the catheter tip. The catheter is inserted into the bladder via the urethra and fixed in place by blocking the balloon in the urinary bladder. The blocking balloon is filled with sterile water with a syringe via the check valve through one course, and urine is drained through the second course. Urine bags with a funnel attachment can be connected to the funnel port. The Charrière sizes (CH) of the catheters are color-coded on the check valve according to their size.

2-Way SERVOCURA® Foley Catheter

Nelaton balloon catheter: Cylindrical closed tip with two opposing eyes.

Tiemann balloon catheter: Curved tapered hollow tip slightly thickened at the end with one eye.

Length 40 cm / filling volume of the blocking balloon 5 ml to 10 ml.

Order Information

Tip	Size (CH)	Color	Total length	Max. filling volume of balloon	Order No.
Nelaton Tip	12	White	40 cm	5-10 ml	32302
Nelaton Tip	14	Green	40 cm	5-10 ml	32303
Nelaton Tip	16	Orange	40 cm	5-10 ml	32304
Nelaton Tip	18	Red	40 cm	5-10 ml	32305
Nelaton Tip	20	Yellow	40 cm	5-10 ml	32306
Nelaton Tip	22	Purple	40 cm	5-10 ml	32307
Tiemann Tip	12	White	40 cm	5-10 ml	32308
Tiemann Tip	14	Green	40 cm	5-10 ml	32309
Tiemann Tip	16	Orange	40 cm	5-10 ml	32310
Tiemann Tip	18	Red	40 cm	5-10 ml	32311
Tiemann Tip	20	Yellow	40 cm	5-10 ml	32312

7 Use

Note

- Please observe the hygienic instructions of the Robert Koch Institute (RKI) for these balloon catheters: see (https://www.rki.de/DE/Content/Infekt/Krankenhaushygiene/Pflege/Heime_node.html).
- Insert the catheter slowly. If the catheter is inserted too quickly, bleeding may occur or the patient may experience shock.
- Make sure that the catheter is not inserted unintentionally into the vagina.

1.5 Use

- Wash and disinfect hands thoroughly and use sterile single-use gloves.
- Check if the sterile packaging and catheter remain undamaged (visual inspection).
- Disinfect the urethral opening (observe the exposure time!).
- Remove the catheter from the sterile packaging.
- Carefully open the protective foil at the catheter tip and carefully push the catheter out.
- Then moisten the catheter with lubricant (glycerine or lubricant for catheterization) while avoiding contamination. Do not block the eyes.
- Gently and slowly insert catheter into the urethra immediately after applying lubricant until the balloon sits securely within the bladder.
- Inject 5-10 ml of sterile water solution into the check valve with a syringe (luer) to block the balloon.
- The catheter should not accidentally slip out.
- Turn the catheter carefully to ensure that the balloon sits completely in the bladder.
- Disinfect the funnel connection of the catheter, then connect it to the connecting tube of a urine bag and secure the connection.
- If the urine does not come out, check to see if the catheter is blocked or bent.
- For removal, completely empty the balloon via the check valve and slowly and carefully pull the catheter out of the urethra and dispose of it as non-reusable medical waste.

EN

Change intervals according to the recommendation of Robert Koch Institute (RKI):

„Indwelling bladder catheters should not be changed routinely at fixed intervals, but as needed based on individual considerations (e.g., incrustation, obstruction, contamination).“

Procedure if catheter is difficult to remove (balloon cannot be unblocked).

Follow the physician's instructions and follow the following steps if the catheter is difficult to remove.

- Inject 1 ml of sterile water through the check valve and drain the water slowly.
- If the sterile water cannot be injected or drained:
 1. Cut off the check valve and let the water run out.
 2. If water does not drain after cutting off the valve, disconnect the Y-piece and let the water run out.
 3. If these measures fail to drain the water, the position of the balloon must be determined by ultrasound so that the water can be drained by puncturing the balloon.

8. Storage, Transport and Lifetime

Store or transport under dry conditions, free of dust and protected from sunlight and light. Avoid prolonged exposure to extreme temperatures and ultraviolet light. The catheters have a maximum shelf life of 3 years in the intact package and in compliance with the storage conditions and may be used on a patient for a maximum period of 29 days if the instructions for use are observed.

9. Disclaimer

In case of non-observance of the Instructions for Use or in case of manipulation of the product, the manufacturer does not assume any liability if damages occur.

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